

ANTIGEN RAPID TEST BIOSYNEX COVID-19 AG BSS



ANTIGEN RAPID TEST BIOSYNEX COVID-19 AG BSS

- Immunochromatographic test in cassette form, for rapid diagnosis of Covid-19
- Kit format, requires no additional material or equipment
- From nasopharyngeal swabs
- 96% sensitivity and 100% specificity
- No cross-reactivity with human corona viruses (229E, OC43, NL63 and HKU1), influenza A and B and RSV A and B
- Result in 15 minutes
- CE IVD certified

Kit contents:

- 25 test cassettes
- 2 vials with extraction buffer
- 25 sterile swabs
- 25 extraction tubes
- Stand
- Instruction manual



Cat. Num.	Description	Units/box
674200	Antigen rapid test Biosynex covid-19 Ag BSS	25 cassettes

RESULT




Positive




Negative


PROCEDURE

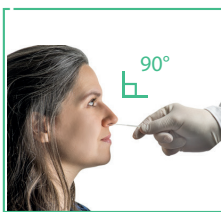


Insert the swab **horizontally** into the nasal cavity

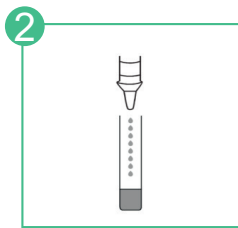


Add 10 drops (0.3 ml) of extraction buffer

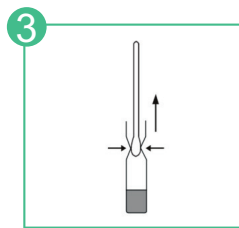

Discharge the swab into the tube by **rotating** the swab for **about 1 minute**

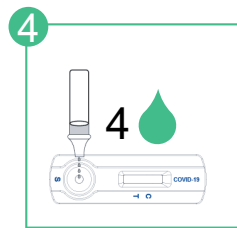

Place the test on the tube and pour **4 drops** of the solution into the well S

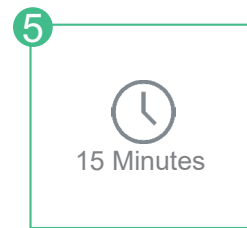

Read the result after **15 minutes**. Do not interpret the result after 20 minutes

For the complete procedure read the manual